


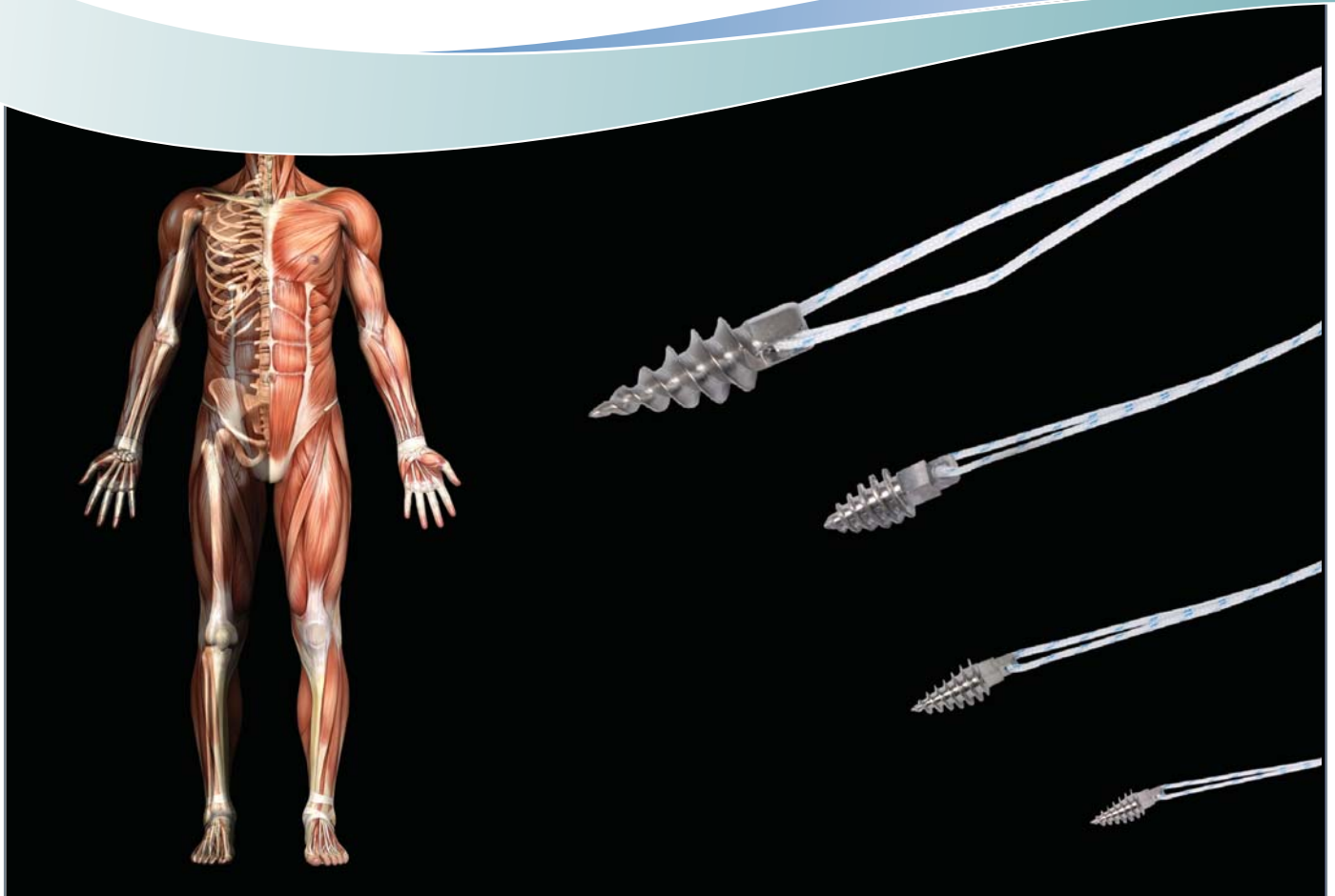

Suture Anchor System



Technology Serving Technique

SUPERIOR DESIGN AND SUTURE MATERIAL OFFERING STRENGTH AND SIMPLICITY

reNOVO™ is a comprehensive line of titanium, self-tapping suture anchors pre-loaded with high strength ultraFibre™ Polyethylene sutures. The reNOVO product line offers a wide range of sizes and configurations to support both arthroscopic and open procedures. A predrill set is available for hard bone. All instruments have comfortable grip handles with compartments for suture and needle storage. With superior design and suture material, reNOVO offers strength and simplicity.

5.5mm Anchor with needles



HIGH STRENGTH POLYETHYLENE ULTRAFIBRE™ SUTURES


- Increased tensile strength
- Increased knot strength
- Excellent handling properties
- Increased resistance to fraying
- Smooth eyelet design
- Limits suture damage and provides better suture manipulation
- Shaft markings for directional eyelet placement and depth

DIRECTIONS FOR USE


1. Pre-drill to depth established by the stop on the tool.
2. Remove the plastic cap from the 2, 3, 4mm anchors by pinching the distal end of the cap and turning while pulling. Lift the plastic lid for the 5.5mm anchors. Be careful not to dislodge the anchor while removing the cap or lid.
3. Screw anchor to the transition point of the cylinder and cone (2, 3, 4mm anchors) or laser line (5.5mm anchors).
4. Remove the plastic suture retaining sleeve from the shaft.
5. Remove the white cap to expose needles (2, 3mm anchors) or remove top blue lid (4, 5.5mm anchors).
6. Pull shaft away from anchor.

Note: If the anchor is not to depth; pull tension on the sutures, insert sutures into slot, move driver shaft back over the eyelet and turn handle to advance the anchor depth.

2mm, 3mm Anchors

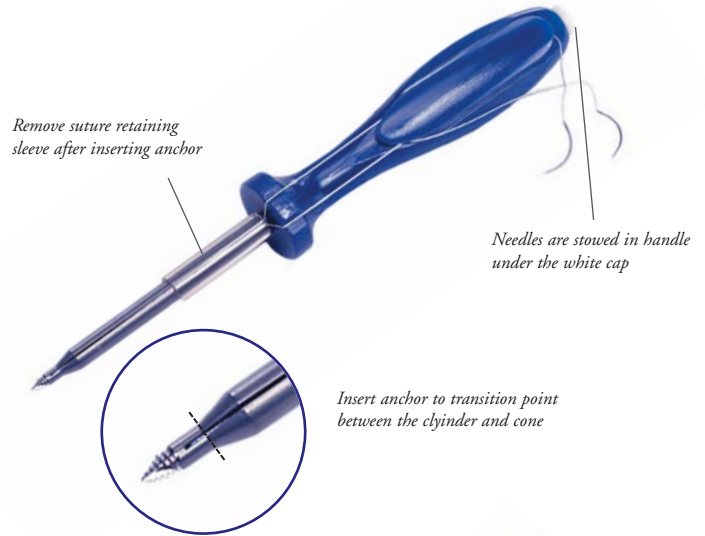


5.5mm Anchors

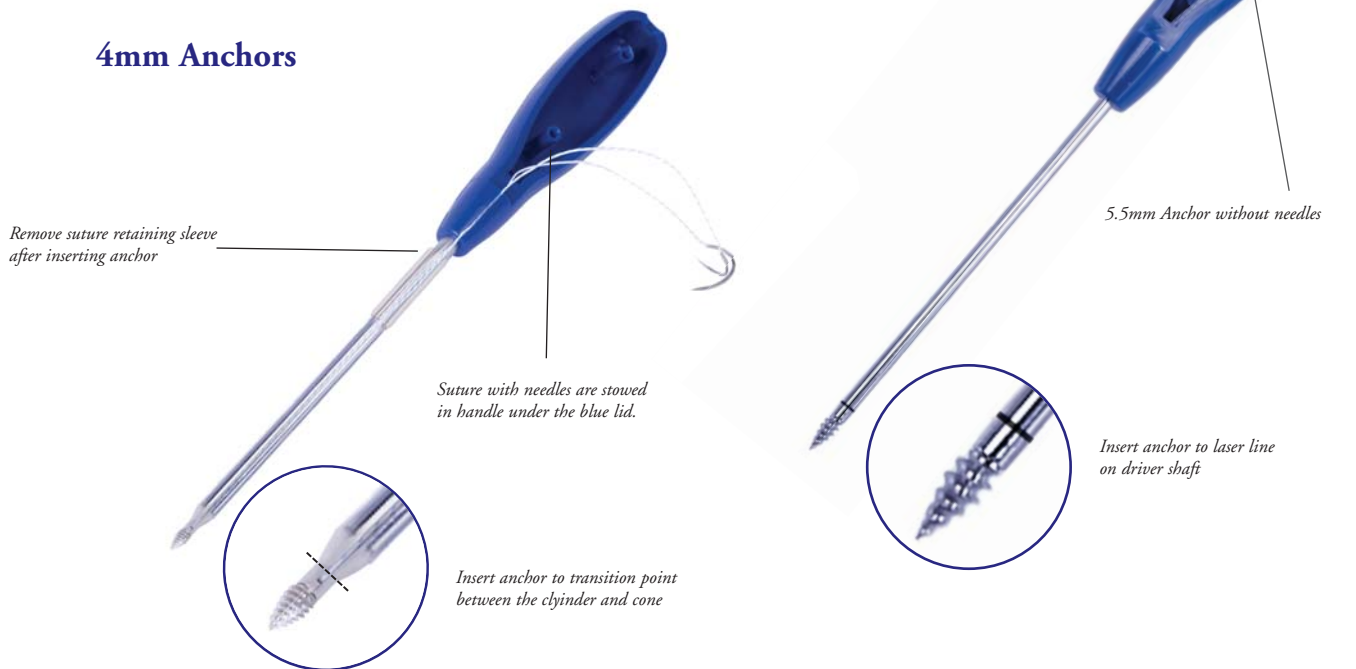

Product Line Information

Reference	Anchor Diameter (mm)	Anchor Length (mm)	Qty. of Sutures	USP (Ref.) Sutures Size/Length	Needle Type	Length/Arc
100-2001	2.0	5.2	1	# 3-0 50 cm	Taper	20mm/135°
100-3001	3.0	7	1	# 2-0 50 cm	Taper	20mm/135°
100-4001	4.0	10.2	1	# 2 50 cm	Taper	25mm/180°
					Cut	35mm/180°
100-5502	5.5	15.7	2	# 2 50 cm	Taper	25mm/180°
100-5502X	5.5	15.7	2	# 2 90 cm	Cut	35mm/180°

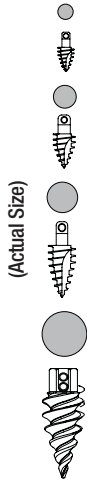

575A Virginia Drive • Fort Washington, PA 19034 USA
P: 215.310.9534 • F: 215.660.5014 • www.CEOrtho.com